


Discard EBM

Scenario


An EBM batch needs to be discarded due to contamination / spillage.

You have logged into the handheld device.

Instruction

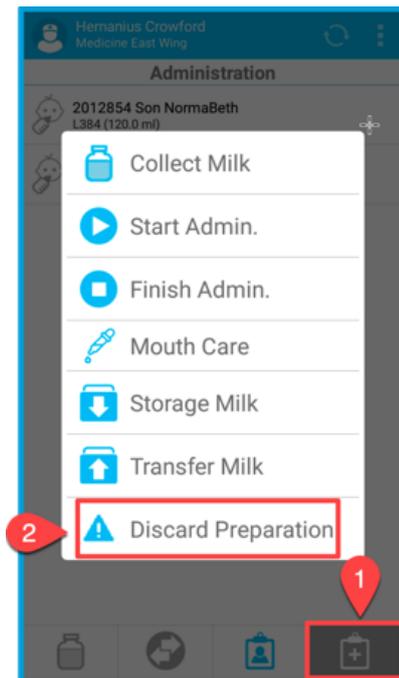

1. Open the options menu.
2. Select the **Discard Preparation** option.

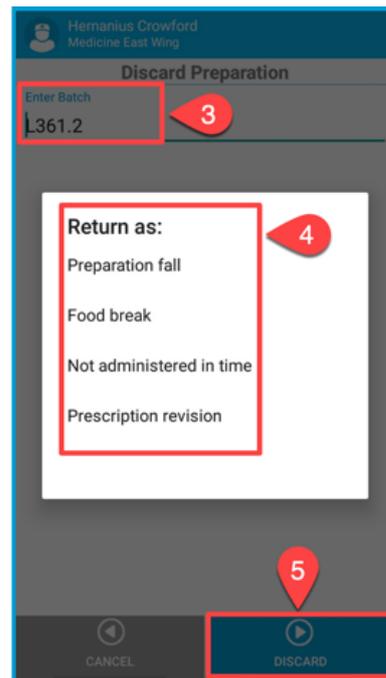

3. Scan the EBM barcode.
4. Select the reason.
5. Press on **Discard** to commit process.
6. After the server receives the data, you will receive a confirmation notice, not shown here. Press **OK**. If you did not receive confirmation, then the server did not receive the data and you will need to **Discard** again.

i Discard removes batch from availability but the system will retain the information that the batch was discarded. This can be viewed on Milktrac's PC app under "Batches".